


*GO*₂ USER MANUAL




Please read this manual before using the GO2.

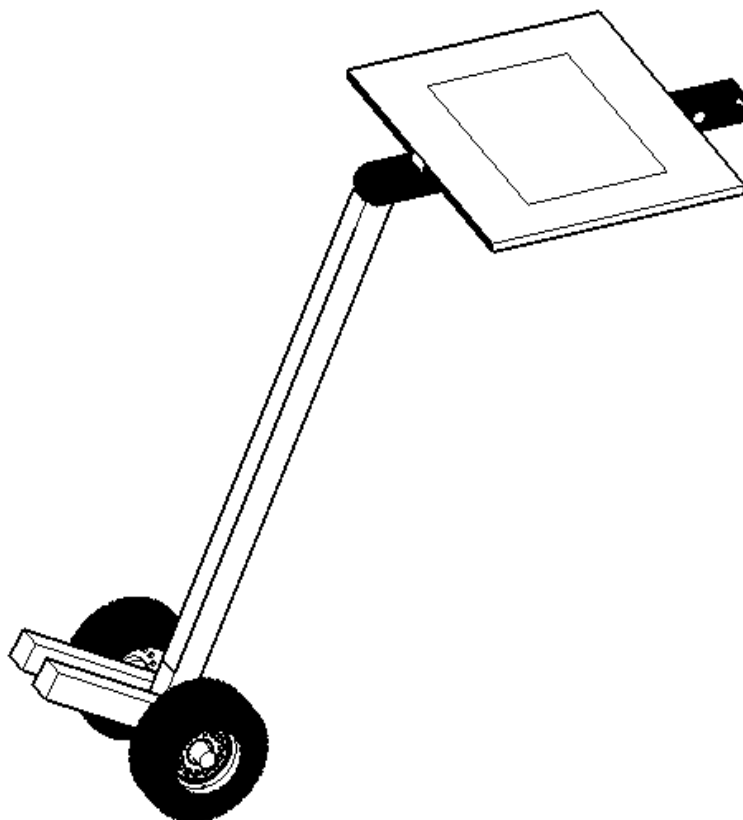

PARTS


WHEELS

The GO2 is one complete accessory. Two removable wheels are attached to the main accessory.

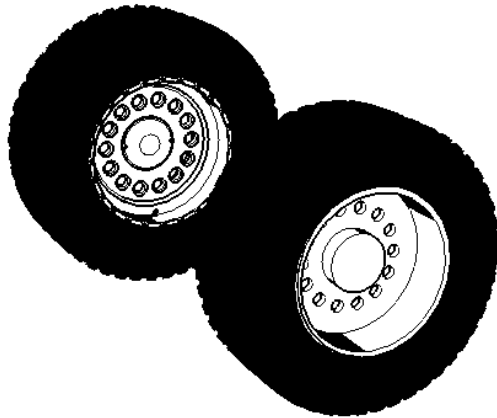


RESPIRATOR HOLDER

The respirator holder is welded on top of the accessory to carry a CPAP respirator. This holder can support up to 25 lbs. It is not recommended to exceed the limit which can cause a rupture in the structure.

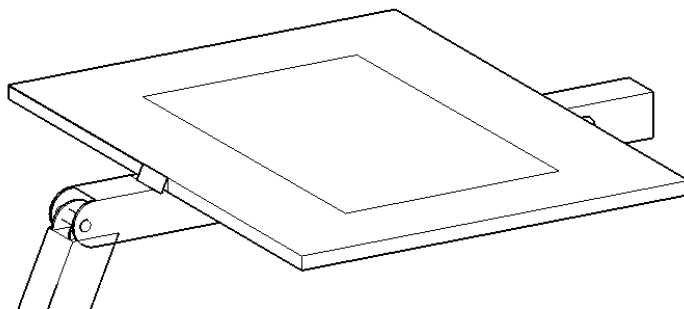


OXYGEN HOLDER

The oxygen holder supports all portable oxygen tanks up to Size E. The weight limit for the oxygen holder is 50 lbs. Do not exceed the weight limit. Fracture of the structure can occur.

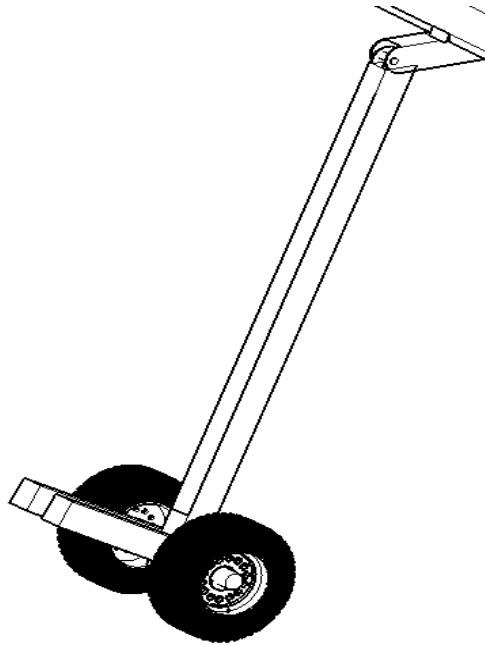

HITCH CUBE

The hitch cube is designed to attach to any 1" hitch cube available at the back of the wheelchair.

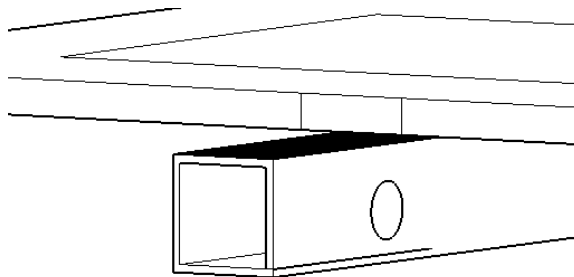

SAFETY PRECAUTIONS

Do not exceed weight limit.

When attaching to chair, cautiously remove fingers from chair.


Keep away from children.

Before using, ensure all wheels are tightened.

Do not drive recklessly; accessory might break.


INSTALLATION

A. Numbers 1, 2 and 3 are for adjusting the chair before installing the GO2. Please make sure the chair suits your needs and is capable of holding the GO2.

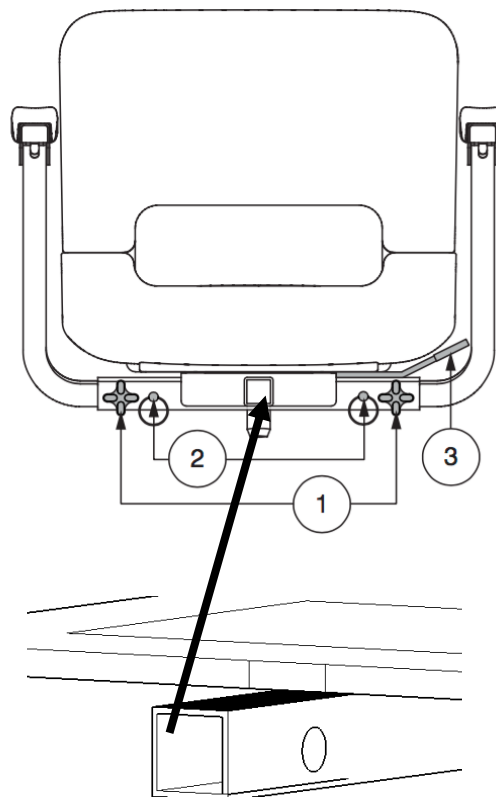

B. Insert the cube hitch into the 1" hitch of the chair. Make adjustments by rotating the adjustable screw, clockwise.

C. Arrange the oxygen tank on the oxygen tank holder. Adjust straps with buckles according to tank size.

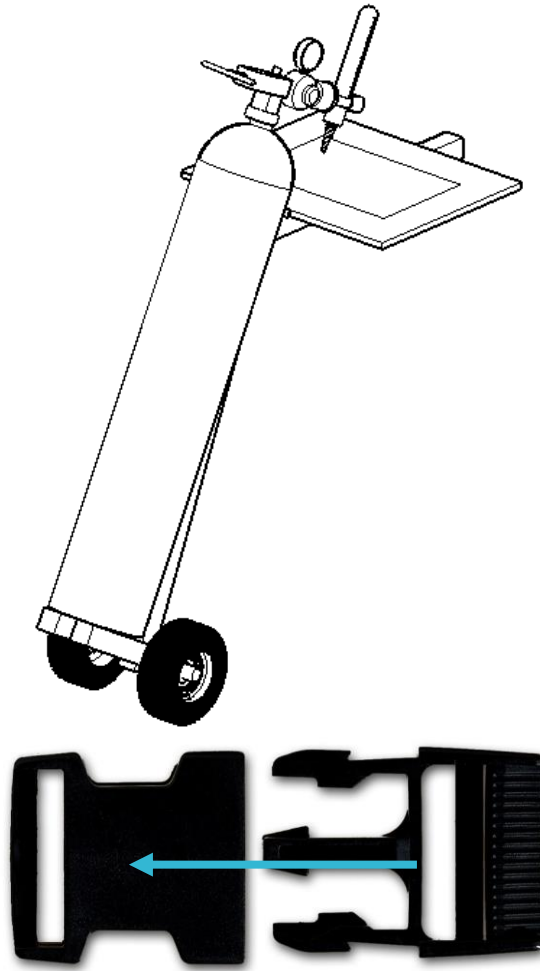

D. Arrange the CPAP respirator onto the respirator holder. Secure respirator with straps. Tighten the buckle and adjust the straps for secure mobility.

References

Buckles: <http://www.shann.co.nz/product/category/marine/buckles-mar>

Chair: <http://www.pridemobility.com/gogo/gogoelitetravellerplus.asp>

Drawing: Sketch Up Make 2017